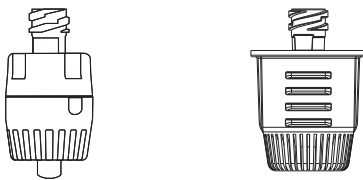
Arisure® Closed Male Luer

REF YM007

REF YM058

EN Arisure Closed Male Luer	EL Κλειστός αρσενικός σύνδεσμος luer του Arisure
FR Luer mâle fermé Arisure	ET Arisure Luer-tüüpi suletud pistik
ES Conector Luer macho cerrado Arisure	HU Arisure Zárt Apa Luer
IT Connettore luer maschio chiuso Arisure	AR وصلة لوير Arisure نكر مغلقة
DE Arisure geschlossener männlicher Luer	LV Arisure slēgtais Male Luer
SV Arisure Stängd Han Luer	PL Zamknięty luer męski Arisure
NL Arisure Gesloten Mannelijke Luer	RO Arisure Închis Barbat Luer
NO Arisure stengt mannlig luer	RU Закрыты льеровскі конус Arisure
DA Arisure lukket han-Luer	SK Uzavretý kuželový uzáver Arisure
FI Arisuren suljettu uros-luer-sovitin	SL Arisure zaprt moški nastavek luer
PT Arisure Luer Macho Fechado	SR Arisure zatvoreni muški luer
TR Arisure Kapalı Erkek Luer	ZH Arisure 封闭式公螺旋口
CS Uzavřený samčí luer Arisure	

Rx Only

Yukon Medical, LLC
4021 Stirrup Creek
Durham, NC 27703 USA

EC REP Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands

CE 2797

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<p>Does not contain DEHP*</p>  <p>DEHP-FREE</p> <p>SS-EN 15986:2011*</p>	<p>Indicates the medical device does not contain bis (2-ethylhexyl) phthalate (DEHP).</p>
<p>Not made with natural latex rubber</p>  <p>5.4.5</p>	<p>Indicates dry natural rubber latex is not a material of construction within the medical device or the packaging of a medical device.</p>
<p>For prescription use only**</p> <p>Rx Only</p> <p>CFR 801.109**</p>	<p>Caution: Federal Law restricts this device to sale by or on the order of a physician.</p>
<p>Non-pyrogenic</p>  <p>5.6.3</p>	<p>Indicates a medical device that is non-pyrogenic.</p>
<p>Do not use if package is damaged</p>  <p>5.2.8</p>	<p>Indicates a medical device that should not be used if the package has been damaged or opened.</p>
<p>Catalog Number</p>  <p>5.1.6</p>	<p>Indicates the manufacturer’s catalogue number so that the medical device can be identified.</p>

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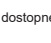
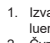
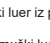
Toto zariadenie neobsahuje polyvinylchlorid (PVC).

<p>Consult instructions for use</p>  <p>5.4.3</p>	<p>Indicates the need for the user to consult the instructions for use.</p>
<p>Do not re-use</p>  <p>5.4.2</p>	<p>Indicates a medical device that is intended for one use, or for use on a single patient during a single procedure.</p>
<p>Use-by date</p>  <p>5.1.4</p>	<p>Indicates the date after which the medical device is not to be used.</p>
<p>Batch Code</p>  <p>5.1.5</p>	<p>Indicates the manufacturer’s batch code so that the batch or lot can be indentified.</p>
<p>Sterilized using irradiation</p>  <p>5.2.4</p>	<p>Indicates a medical device that has been sterilized using irradiation.</p>
<p>Caution</p>  <p>5.4.4</p>	<p>Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself.</p>

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Ta naprava ne vsebuje polivinil klorida (PVC).

<p>Manufacturer</p>  <p>5.1.1</p>	<p>Indicates the medical device manufacturer, as defined in EU Directives 90/385/EEC, 93/42/EEC and 98/79/EC</p>
<p>Authorized representative in the European Community</p>  <p>5.1.2</p>	<p>Indicates the Authorized representative in the European Community.</p>

All symbols in this table are from *ISO 15223-1:2012 - Medical Devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1 General requirements*, except if specifically noted

*This symbol is from *SS-EN 15986:2011 - Symbol for use in the labelling of medical devices - Requirements for labelling of medical devices containing phthalates*

**This symbol is from *CFR 801.109 - Code of Federal Regulations Title 21 Chapter I Subchapter H Part 801 Section 109 Prescription Devices*

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Yukon Medical and ViaLok are trademarks or registered trademarks of Yukon Medical, LLC.