
Neutral Valve

REF YM004

EN Neutral Valve	EL Ουδέτερη βαλβίδα του	DEHP-FREE	Rx Only
FR Valve Neutre	ET Neutraalne Klapp		
ES Válvula Neutra	HU Semleges Szelep		
IT Valvola Neutra	AR صمام المحايد		
DE Neutrales Ventil	LV Neutrālais Vārstis		
SV Neutral Ventil	PL Zawór neutralny		
NL Neutrale Klep	RO Valve neutru		
NO Nøytral Ventil	RU Нейтральный клапан		
DA Neutral Ventil	SK Neutrálny Ventil		
FI Neutraali Venttiili	SL Nevtralni Ventil		
PT Válvula Neutra	SR Neutrālni Ventil		
TR Nötr Valf	ZH 中立加药口		
CS Neutrální Ventil			

Yukon Medical is a trademark or registered trademark of Yukon Medical, LLC.
www.yukonmedical.com

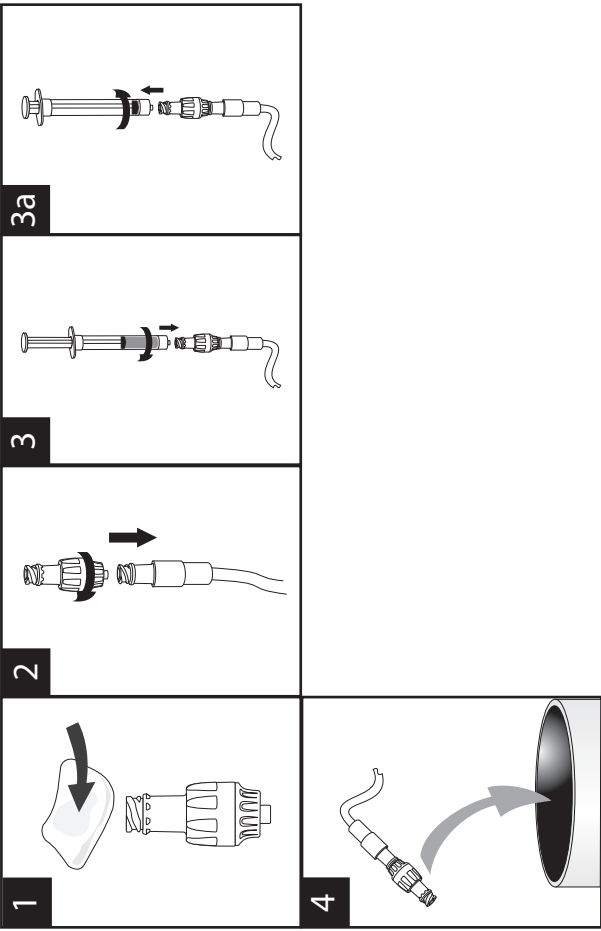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EN
<p>Indications for Use</p>

The Neutral Valve is a single use, sterile, non-pyrogenic device intended for use as an accessory to intravascular administration sets for the administration or withdrawal of fluids from a patient through a cannula placed in the vein or artery. The Neutral Valve may be used with low-pressure power injectors up to 300 psi.

<p>Precautions:</p>
<ul style="list-style-type: none">Do not spray package with cleaning agents as damage to labeling may occur. Do not use if package is open or damaged Do not use blunt cannula or needles to access the Neutral Valve. Discard protective cap (packaging) after removing the device. Do not cap Neutral Valve, device is closed. Not approved for use with alcohol-impregnated disinfection caps. For use with ISO-594 compatible locking luers. Do not leave slip luers attached to the device unattended. Replace device every 96 hours or after 128 actvations of Neutral Valve, whichever comes first.

<p>Directions</p>
<ol style="list-style-type: none">Prior to every access, swab the top of the Neutral Valve with 70% isopropyl alcohol. Disinfect the valve by thoroughly scrubbing the top of the valve with an alcohol pad for at least 15 seconds using circular motions. Shift to a new area of the pad at least once during the 15 second scrubbing period. Allow to dry (approximately 30 seconds). Attach male luer of Neutral Valve to a mating female luer on an administration set. NOTE: Prime the valve and expel air as needed. To attach a fully primed syringe or administration set for administration or aspiration, grasp the Neutral Valve around the device body and firmly push and twist male adapter straight into valve until secure. Post administration, disconnect the Neutral Valve by grasping the valve and twisting the syringe or administration set away from the Neutral Valve until loose. NOTE: Flush the Neutral Valve with saline or in accordance with facility protocol after each use. Dispose the Neutral Valve in accordance with facility protocol and applicable local, state, and federal laws and regulations.
<p>This device does not contain Polyvinyl Chloride (PVC).</p>

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<div>Does not contain DEHP*  DEHP-FREE SS-EN 15986:2011*</div>	Indicates the medical device does not contain bis (2-ethylhexyl) phthalate (DEHP).
<div>Not made with natural latex rubber  5.4.5</div>	Indicates dry natural rubber latex is not a material of construction within the medical device or the packaging of a medical device.
<div>For prescription use only** Rx Only CFR 801.109**</div>	Caution: Federal Law restricts this device to sale by or on the order of a physician.
<div>Non-pyrogenic  5.6.3</div>	Indicates a medical device that is non-pryogenic.
<div>Do not use if package is damaged  5.2.8</div>	Indicates a medical device that should not be used if the package has been damaged or opened.
<div>Catalog Number  5.1.6</div>	Indicates the manufacturer's catalogue number so that the medical device can be identified.

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Toto zariadenie neobsahuje polyvinylchlorid (PVC).

<div>Consult instructions for use  5.4.3</div>	Indicates the need for the user to consult the instructions for use.
<div>Do not re-use  5.4.2</div>	Indicates a medical device that is intended for one use, or for use on a single patient during a single procedure.
<div>Use-by date  5.1.4</div>	Indicates the date after which the medical device is not to be used.
<div>Batch Code  5.1.5</div>	Indicates the manufacturer's batch code so that the batch or lot can be indentified.
<div>Sterilized using irradiation  5.2.4</div>	Indicates a medical device that has been sterilized using irradiation.
<div>Caution  5.4.4</div>	Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself.

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Ta naprava ne vsebuje polivinil klorida (PVC).

<div>Manufacturer  5.1.1</div>	Indicates the medical device manufacturer, as defined in EU Directives 90/385/EEC, 93/42/EEC and 98/79/EC
<div>Authorized representative in the European Community  5.1.2</div>	Indicates the Authorized representative in the European Community.

All symbols in this table are from *ISO 15223-1:2012 - Medical Devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1 General requirements*, except if specifically noted

*This symbol is from *SS-EN 15986:2011 - Symbol for use in the labelling of medical devices - Requirements for labelling of medical devices containing phthalates*

**This symbol is from *CFR 801.109 - Code of Federal Regulations Title 21 Chapter I Subchapter H Part 801 Section 109 Prescription Devices*

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