

IV ADMINISTRATION SET UP

STEP 1

Holding the Arisure® Closed Male Luer (CML) firmly around the device body, attach to the syringe or distal end of the administration set if not already attached. Avoid touching the Luer access points.



STEP 2

Prior to every access swab the needle-free valve and the Arisure® Closed Male Luer (CML) with 70% Isopropyl alcohol (IPA) in a circular motion, using aseptic technique or follow facility protocol.

STEP 3

To connect to the Arisure® Neutral Valve (ANV), *BD™ SmartSite™, *Baxter ONE-LINK or CLEARLINK connectors, grasp the Arisure® (CML) valve around the body of the device and align with the body of the Arisure® ANV/SmartSite/ONE-LINK/CLEARLINK connector. Make sure to avoid touching the Luer access points. After aligning, firmly push the Arisure® (CML) stem into the Arisure® ANV/SmartSite/ONE-LINK/CLEARLINK connector until the threads connect. Secure the Arisure® (CML) valve to the device.



NOTE: Do not attach at an angle. Optimal connection is a straight on connection where the locking threads come to a physical stop. This is required to maximize the performance of the device.

IV PUSH SET UP

STEP 1

Prior to every access, swab the needle-free valve and the Arisure® Closed Male Luer (CML) with 70% Isopropyl alcohol (IPA), using aseptic technique or follow facility protocol.

STEP 2

To connect to the Arisure® Neutral Valve (ANV), SmartSite™, ONE-LINK or CLEARLINK connectors, grasp the Arisure® (CML) valve around the body of the device and align with the body of the Arisure® ANV/SmartSite/ONE-LINK/CLEARLINK connector. Make sure to avoid touching the Luer access points. After aligning, firmly push the Arisure® (CML) stem into the Arisure® ANV/SmartSite/ONE-LINK/CLEARLINK connector until the threads connect. Secure the Arisure® (CML) valve to the device.



NOTE: Do not attach at an angle. Optimal connection is a straight on connection where the locking threads come to a physical stop. This is required to maximize the performance of the device.

DISCONNECTING

- » When disconnecting an Arisure® Closed Male Luer (CML) valve with an attached administration set, ensure that you first close all clamps on your administration set or follow facility protocol.
- » Grasp the closed Arisure® Closed Male Luer (CML) valve around the device body and twist counter clockwise to disconnect from the Arisure® Neutral Valve (ANV), *BD™ SmartSite™, *Baxter ONE-LINK or CLEARLINK connectors.
- » After medication is administered, dispose of Arisure® products in accordance with your facility's policy and procedures.
- » Do not disconnect any of the Arisure® components prior to disposal.

NOTE: The Arisure® Closed Male Luer (CML) should not be capped with any device other than the yellow dust cap. Dead end or alcohol swab caps will engage the device resulting in potential hazardous drug exposure. Please see the Instructions for Use for a complete listing of the indications, contraindications, warnings and precautions.

**SmartSite is a trademark of Becton, Dickinson and Company, its subsidiaries or affiliates. Yukon Medical Inc. is not affiliated with Becton, Dickinson and Company, *ONE-LINK AND CLEARLINK are trademarks of Baxter International Inc, its subsidiaries or affiliates. Yukon Medical Inc. is not affiliated with Baxter International Inc.*

YMIV0003revMAR23

**SCAN TO WATCH THE
ARISURE® NURSING
IN-SERVICE VIDEO NOW**



Exclusive U.S. Distributor:
Progressive Medical, Inc. (PMI)
997 Horan Drive • Fenton, MO 63026
Ordering Information:
Tel: 314.961.5786 • Fax: 314.961.4535
cs@progressivemedinc.com • progressivemedinc.com