

INFORMATION INSERT

Instructions for use of this Operative Cholangiogram Catheter

READ AND BECOME FAMILIAR WITH ALL INSTRUCTIONS AND CAUTIONS BEFORE USING THIS PRODUCT

INDICATIONS FOR USE: To facilitate the introduction of contrast media into the gallbladder to obtain a cholangiogram during laparoscopic or open cholecystectomy.

CONTRAINDICATIONS: This device is not intended for use except as indicated.


1. Prior to insertion of the catheter, ensure that the system is free of air.
2. Introduce the ORCA into a clamp (not provided).
3. Insert the catheter needle tip directly into the gallbladder.
4. Following the cholangiogram, withdraw the catheter.


Caution: Federal (USA) law restricts this device to sale by or on the order of a physician. DO NOT alter this device.

Distributed by:
Progressive Medical Inc.
997 Horan Drive
Fenton, MO 63026
800-969-6331

Ref: PMIORCA

Not made
with natural
rubber latex

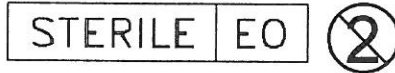

 Ranfac Corp.
30 Doherty Ave.
Avon, MA 02322
USA

09644-03M Rev A