

## PRODUCT INFORMATION DATA SHEET

The following information should be read before using this device

Product No: PMI-10



Luer Lock Connector

Product No: PMI-102




Quick Connector

Product No: PMI-6000



Funnel Connector

Product No: PMI-1522

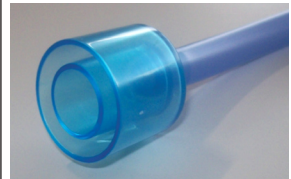

15 / 22 mm Connector

### **Description:**

The Disposable 10ft. Insufflation Tubing Set consists of one ULPA filter with DEHP-free tubing. Latex Free.

### **Intended Use:**

The Insufflation Tubing Set is used to transport filtered CO<sub>2</sub> from the insufflator into the patient cavity for laparoscopic procedures.

### **Note:**

It is supplied sterile (*unless package is damaged or opened*). All contents are ETO sterilized.

### **Instructions for use:**

1. Peel the Pouch to access sterile product.
2. Place Insufflation Tubing on sterile field.
3. Attach distal luer lock connector to veress needle stopcock.
4. Attach specialized connector to insufflator.
5. To allow flow of CO<sub>2</sub>, turn on insufflator and turn stopcock on veress needle.

### **Cautions / Warnings:**

The Insufflation Set is designed for single use only and cannot be reused or resterilized.