



ENDOSCOPIC DISSECTOR INSTRUMENT WITH KITTNER TIP (5MM X 40CM)

ITEM NUMBER: PMILK40 INSTRUCTIONS FOR USE

DESCRIPTION

The Laparoscopic Kittner is comprised of a fiberglass handle with a radiopaque cotton tip.

INDICATIONS

The Laparoscopic Kittner is available in a 5mm size. Indications for Use:

- Gynecological Laparoscopy
- Laparoscopic Cholecystectomy
- Other Laparoscopic Procedures

CONTRAINDICATIONS

This device is not intended for use except as indicated. It is not intended for use when endoscopic techniques are contraindicated.

INSTRUCTIONS FOR USE

1. Insert the Laparoscopic Kittner through a properly placed laparoscopic cannula.
2. The Laparoscopic Kittner is used for blunt dissection or as required.
3. Properly dispose of product upon completion of the procedure.

CAUTIONS

- This instrument is not indicated for use by any person other than surgeons trained in endoscopic procedures.
- A thorough understanding of the principles and techniques involved in laparoscopic laser and electrosurgical procedures is essential to avoid shock and burn hazards to both patient and medical personnel and damage to the device or other medical instruments.
- This instrument is for single patient use and is not intended for reuse or re-sterilization.

Distributed by:

Progressive Medical, Inc.

997 Horan Drive

Fenton, Mo 63026

cs@progressivemedinc.com