

## PRODUCT INFORMATION DATA SHEET

The following information should be read before using this device

**Product No: PMI-500**



**Description:**

Disposable, sterile heating device.

**Intended Use:**

Pre-warms scopes to body temperature to keep the scopes free from condensation.

**Note:**

It is supplied sterile (*unless package is damaged or opened*). All contents are ETO sterilized.

**Instructions for use:**

1. Place scope warmer on a towel on the sterile field.
2. Activate the Scope Warmer ***3-5 minutes before actual use.***
3. Insert one or two scopes, any size or length.
4. Side One: Flex metal disk back and forth once.
5. Side Two: Flex metal disk back and forth once.
6. Bag will start to “cloud up” & begin heating immediately.
7. Scope will start warming to body temperature in less than 2 minutes (place another towel on top of warmer).

**Cautions / Warnings:**

Scope Warmers are designed for single use only and cannot be reused or resterilized.