



REORDER NO. :

PMIFR100

**Administration set
with Flow Controller**

Disposable, Single Use Device. Priming Volume:
28.2 ml (Approx.), Length: 100" (254cm) , Universal
Spike Drip Chamber (20 drops/ml), upper pinch clamp,
dial type flow controller, rotating male luer lock.

DEHP FREE, LATEX FREE

Rx Only (USA)

DIRECTIONS: USE ASEPTIC TECHNIQUE

1. Open package at designated end and remove set.
2. Close clamp on set and turn flow controller to OFF position. Remove end cap on spike and insert into solution container.
3. Squeeze drip chamber to approximately 1/2 full.
4. Turn flow controller to full open position and purge all air from the set.
5. When all air is flushed, turn controller dial to OFF position.
6. Remove end cap and attach male luer of primed set to desired venous access device.
7. Turn dial to desired rate. Dial numbers are approximate. Confirm drip rate by using a timer or a watch with a second hand.
8. To stop flow, close clamp and turn flow controller dial to OFF position.

Federal (USA) law restricts this device to sale by or on the order of a physician or other licensed practitioner.

LOT. : 20170714

EXP. : 2020-07-13

Manufactured in China for
PROGRESSIVE MEDICAL, INC.
997 HORAN DRIVE FENTON,
MISSOURI 63026-2401



Sterile R