




PROGRESSIVE MEDICAL

PMI Huber Safety Infusion Set INSTRUCTIONS FOR USE

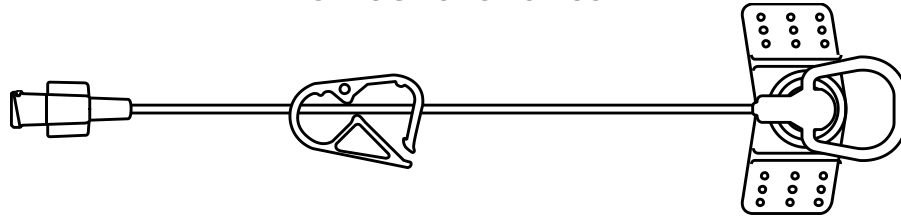

DESCRIPTION

Sterile, PMI Huber Safety Infusion Sets are available in different needle lengths and gauges.

12 per box of: • Straight Huber Set • Y Injection Site Huber Set • Needle-Free Y Huber Set

INDICATIONS FOR USE

The PMI Huber Safety Infusion Set is a safety IV administration set used to access vascular ports. This set is used to administer fluids or to withdraw blood. The PMI Huber Safety Infusion Set facilitates safe removal of the needle by encapsulating the needle within a telescoping plastic cone to help prevent needlestick injuries when using the device.

WARNINGS & PRECAUTIONS

- Always follow standard infection control precautions as specified by the CDC (USA) or local equivalent.
- Leakage may occur with Luer fittings over time or when disconnecting components; use CDC specified precautions to prevent exposure to blood-borne pathogens.
- Verify that the clamp is open and that there are no kinks in the IV line before priming or initiating flow.
- If the telescoping plastic cone safety feature fails to completely cover and protect the contaminated needle during removal, use care in handling the infusion set and immediately dispose of it into a sharps container.

USE

Insertion

1. Prepare port site for sterile needle insertion.
2. Attach a fully primed syringe or administration set to the injection site or Luer hub.
3. Grasp the wings and pinch together upwards, remove the needle cover. Prime and flush the infusion set per institutional protocol.
4. Stabilize the port, and insert the primed needle through the skin and into the port. Ensure correct needle placement by aspirating for blood. Flush per institutional protocol.
5. Fold wings down against the skin and secure the needle to the site following institutional protocol.

Removal

1. Using your non-dominant hand, use two fingers to hold down the wings to the patient and stabilize the port.
2. Grasp the pull-tab with your dominant hand and pull straight up, while the fingers on top of the wings keep the device held down.
3. Pull up until the plastic cone is fully extended and the needle is completely encapsulated.
4. Dispose of as per your institution's sharps disposal guidelines.

Distributed by: Progressive Medical Inc. 997 Horan Drive
Fenton, MO 63026
314-961-5786



Do Not Use If Package Is Damaged



Do Not Re-sterilize

Store in a cool and dry place with
ambient temperatures
(< -15 deg C and > +50 deg C)



CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician.



Read Instructions Before Use
Not Made With Natural Rubber Latex