

Product Description

The Extractor-EZ™ is a disposable instrument designed to enlarge the fascial opening without necessarily enlarging the skin incision to facilitate the removal of specimens in any minimally invasive platform.

Item # PMIXTCR

- A. Blade Button
- B. Blunt Tip Shaft
- C. Retractable Blade

Product Diagram



Indications for Use

This product is designed to facilitate the extraction of any solid organ or specimen from inside a patient, through an existing minimally invasive trocar site, by making an incision in the fascia of the abdominal wall, under direct vision, without loss of visualization or pneumoperitoneum.

Instructions for Use

1. Select a port site with a 10mm or larger trocar for specimen removal.
2. Insert the Extractor-EZ™ into the trocar at the selected site, under direct vision, **with the blade retracted**, until the Retractable Blade extends past the tip of the trocar.
3. Once the Extractor-EZ™ is positioned as desired, with the Retractable Blade past the tip of the trocar, then slide Blade Button forward to expose the Retractable Blade.
4. Create incision(s) in the abdominal wall to enlarge the fascial opening as needed for specimen removal.
5. Once the fascial opening has been adequately enlarged, slide the Blade Button backward to retract the Retractable Blade.
6. Remove the Extractor-EZ™.
7. Remove the specimen.

Warnings

1. This package insert is designed to assist in using the Extractor-EZ™ device and is not a reference to surgical technique.
2. Please read all instructions before using the Extractor-EZ™ device.
3. Failure to properly follow the instructions may lead to surgical consequences.
4. The Extractor-EZ™ is provided STERILE and is intended for single use only. DISCARD AFTER USE. DO NOT RESTERILIZE.
5. If package is damaged or open, do not use the product. Check package for damage prior to use.
6. Laparoscopic procedures should be performed only by a physician having adequate training and familiarity with endoscopic technique.



CAUTION: FEDERAL LAW (USA) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.



CONSULT INSTRUCTIONS FOR USE



STERILIZED USING ETHYLENE OXIDE
STERILITY GUARANTEED UNLESS PACKAGE OPENED OR DAMAGED. DO NOT RESTERILIZE.



DO NOT REUSE. SINGLE PATIENT USE ONLY

NOT MADE WITH NATURAL RUBBER LATEX